

1

## UMBILICAL TAPE

### DESCRIPTION

Umbilical Tape is composed of cotton obtained from North American sources and is available in white.

### INDICATIONS

Umbilical Tape is indicated for use in tying off the umbilical cord of newborn infants.

### ACTIONS/PERFORMANCE

None known.

### CONTRAINDICATIONS

None known.

### WARNINGS

Do not resterilize. Do not re-process. Unit package is sterile unless it has been opened or damaged. Do not use if product packaging is wet. Discard open, unused tape.

In case device is re-used, it is impossible to guarantee its mechanical properties for intended use and there is a risk of cross-infection.

Store at room temperature. Avoid prolonged exposure to elevated temperatures.

Risk of biological contamination; please follow regional or national regulation for elimination of potential contaminated devices, for used tape disposal.

### PRECAUTIONS

In handling this is any other retraction material, care should be taken to avoid damage from handling. Avoid crushing or crimping damage due to application of surgical instruments such as forceps or needle holders.

### ADVERSE REACTION

None known.

3

2

### HOW SUPPLIED

Umbilical tape single use cotton strips are available in sterile, white strands. The tape is supplied sterile in 1/8 inch wide by 18 inches, 30 inches, and 36 inches length in box quantities of 12 and 40 envelopes, and non sterile in 1/8 inch wide by 100 yards.

### STERILITY

Umbilical Tape is sterilized by Ethylene Oxide.

\*CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician or licensed practitioner.

### DISPOSAL


Risk of biological contamination; please follow regional or national regulation for elimination of potential contaminated devices, for used tape disposal.


4

### SYMBOL DEFINITIONS

 Lot Number


 Expiration Date


 Do Not Reuse

 Do Not Resterilize

 See Instructions For Use

 Sterilized By Ethylene Oxide

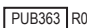
 Keep away from sunlight and heat

 Do not use if package is damaged

 Keep dry

 Manufacturer

 **CP Medical Inc.**  
1775 Corporate Drive, Suite 150  
Norcross, GA 30093 USA

 PUB363 R01

Effective Date 06/25/2020