POLYGLYCOLIC ACID (PGA) COATED BRAIDED WITH CHLORHEXIDINE DIACETATE SYNTHETIC ABSORBABLE SUTURES

DESCRIPTION

Polyglycolic Acid (PGA) is a synthetic, absorbable, sterile surgical suture composed of braided polyglycolic acid and is available both dyed (D&C Violet No. 2) or undyed (milk white). This antibacterial suture is coated with polycaprolactone, calcium stearate and $\leq 60 \text{ µg/m}$ of a common antimicrobial agent. chlorhexidine diacetate

INDICATIONS

Polyglycolic Acid (PGA) is indicated for use in animals in general soft tissue approximation and/or ligation, not for use in cardiovascular and neurological surgery.

ACTIONS

Polyglycolic Acid (PGA) elicits a minimal acute inflammatory reaction in tissues, which is followed by gradual encapsulation of the suture by fibrous connective tissue. Loss of tensile strength and eventual absorption of Polyglycolic Acid (PGA) synthetic absorbable sutures occurs progressively. Absorption begins as a loss of tensile strength followed by a loss of mass. Implantation studies in animals indicate that Polyglycolic Acid (PGA) retains approximately 83% of it's original tensile strength at 14 days post implantation, 52% of it's original tensile strength at 21 days post implantation, and 20% of it's original tensile strength at 28 days post implantation. Absorption of Polyglycolic Acid (PGA) absorbable synthetic suture is essentially complete between 60 and 90 days

This suture is coated with chlorhexidine diacetate, an antimicrobial agent which is intended to reduce or inhibit the colonization of Staphylococcus aureus and Staphylococcus epidermidis, and methicillin-resistant S. aureus (MRSA) and S. epidermidis (MRSE) that may be present at the surgical site.

CONTRAINDICATIONS

This absorbable suture should not be used where extended approximation of tissue is required. The use of this suture may be inappropriate in elderly, malnourished, or debilitated patients, or in patients suffering from conditions which may delay wound healing. For use in animals only, not for use in humans.

A reduction in the colonization of or microbial growth on this device has not been shown to directly correlate with a reduction of infections in patients; acceptable surgical practice should be followed with respect to aseptic technique and the drainage and closure of infected wounds.

ADVERSE REACTIONS

Adverse effects associated with the use of this device include wound dehiscence, failure to provide adequate wound support in closure of the sites where expansion, stretching or distension occur, failure to provide adequate wound support in elderly, malnourished or debilitated patients or in patients suffering from conditions which may delay wound healing, wound infection, minimal acute inflammatory tissue reaction. localized irritation when skin sutures are left in place for greater than 7 days, calculi formation in urinary and biliary tracts when prolonged contact with salt solutions such as urine and bile occurs, and transitory local irritation. Do not use this product on patients with a potential for hypersensitivity or a history of allergies to chlorhexidine diacetate.

Avoid usage in patients that are known to be sensitive to Antibacterial Polyglycolic Acid (PGA) suture.

HOW SUPPLIED

Polyglycolic Acid (PGA) sutures are available in various sizes. PGA sutures are supplied in a wide range of lengths affixed to a diverse assortment of needle types or non-needled

DISPENSING

For best results, pull the suture using a slow, steady pull. If binding occurs, the suture can be removed from the card by opening the card.

CAUTION

Federal (USA) law restricts this device to sale by or on the order of a veterinarian.

FOR VETERINARY USE ONLY

2

Not for use in cardiovascular or neurological surgery.

WARNINGS

Users should be familiar with surgical procedures and techniques involving absorbable sutures before employing polyglycolic acid suture for wound closure, as a risk of wound dehiscence may vary with the site of application and the suture material used.

Do not use if package is open or damaged or if the expiration date has been exceeded. Discard open, unused suture

Do not resterilize; resterilization may alter the physical properties of this suture, which will result in adverse patient reaction

Users should exercise caution when handling surgical needles to avoid inadvertent needle sticks. Discard used needles in a "sharps' container

Store in a cool dry environment

As with any foreign body, prolonged contact of this or any other suture with salt solutions, such as those found in the urinary or biliary tracts, may result in calculus formation.

Acceptable surgical practice should be followed with respect to drainage and closure of contaminated or infected wounds.

The use of supplemental non-absorbable sutures should be considered by the surgeon in the closure of sites which may undergo expansion, stretching, or distention, or which may require additional support as this is an absorbable suture material.

PRECAUTIONS

In handling this or any other surgical suture, care should be taken to avoid damage from handling. Avoid crushing or crimping damage during the use of surgical instruments such as forceps or needle holders

Adequate knot security requires the accepted surgical technique of flat, square ties, with additional throws as warranted by surgical circumstances and the experience of the surgeon

Skin sutures, which must remain in place longer than 7 days may cause localized irritation and should be snipped off or removed

Consideration should be taken in the use of absorbable sutures in tissues with poor blood supply as suture extrusion and delayed absorption may occur.

SYMBOL DEFINITIONS







PUB203 R06 Effective Date 12/01/2023

3

4